



Website




YouTube



Facebook

DISTRIBUTED BY

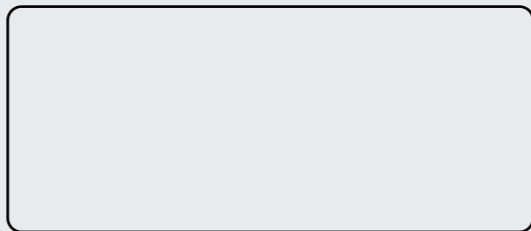

Changzhou Sifary Medcial Technology Co., Ltd
Add: NO.99, Qingyang Road, Xuejia County, Xinbei
District, Changzhou City, 213000 Jiangsu, China
Email: ivy@sifary.com
Web: www.eighteeth.com



TIP BOOK



CATALOG

Scaling

G1

Order Code: 6680523



G1-S

Order Code: 6680617



G2

Order Code: 6680524



G2-S

Order Code: 6680618



G4

Order Code: 6680526



G4-S

Order Code: 6680619



G5

Order Code: 6680527



G5-S

Order Code: 6680620



1

2

3

3

4

Periodontic

P4D

Order Code: 6680631



15

P4-SD

Order Code: 6680637



Endodontic

E3

Order Code: 6680538



16

E3-S

Order Code: 6680657



E4

Order Code: 6680539



17


E4-S

Order Code: 6680658



E5

Order Code: 6680540

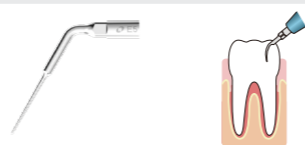

18

Endodontic

E5-S

Order Code: 6680659



18

E7

Order Code: 6680542



19

E7-S

Order Code: 6680660



E9

Order Code: 6680609



20

E9-S

Order Code: 6680615



E3D

Order Code: 6680651



21

E3-SD

Order Code: 6680638



E30

Order Code: 6680611



22

E30-S

Order Code: 6680661



E31

Order Code: 6680612



23

Periodontic

E31-S

Order Code: 6680662



23

E4D

Order Code: 6680652



24

E4-SD

Order Code: 6680639



E5D

Order Code: 6680653



25

E5-SD

Order Code: 6680640



Endodontic

E6D

Order Code: 6680632



26

E6-SD

Order Code: 6680641



E7D

Order Code: 6680633



27

E7-SD

Order Code: 6680642



Cavity Preparation

TC1D

Order Code: 6680648



28

TC1-SD

Order Code: 6680650





SCALING






EMS® G1

Satelec® G1-S

Order Code 6680523

Order Code 6680617


Function Description: All-round removal of supragingival calculus

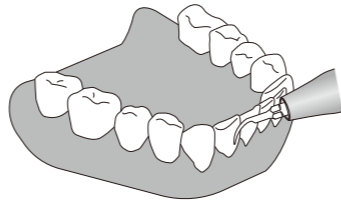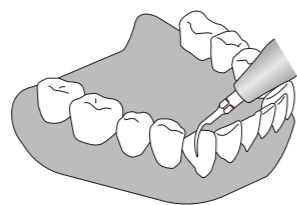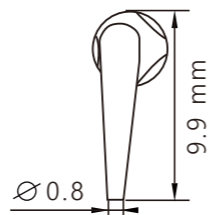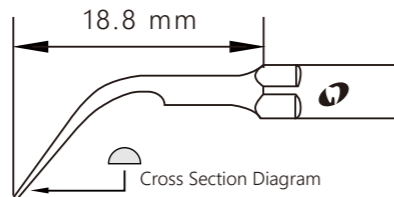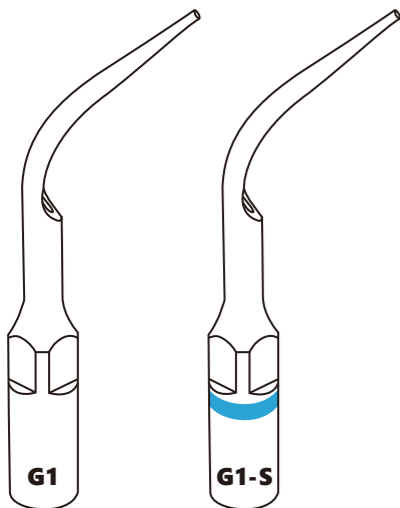

Mode: G
Power: 1-10



Recommended
Service Life
200 Teeth

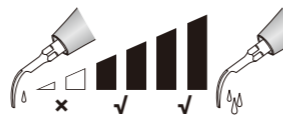

Water Flow Rate
mid&high






EMS® G2

Satelec® G2-S

Order Code 6680524

Order Code 6680618


Function Description: Removal of significant supragingival calculus

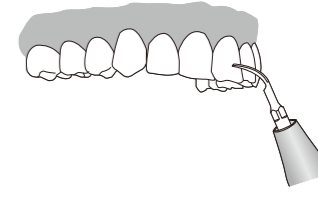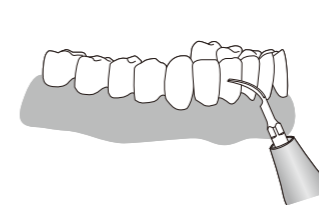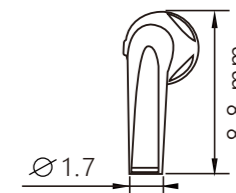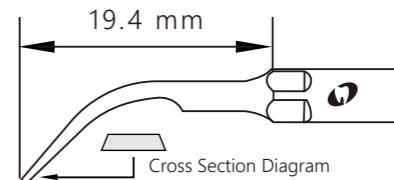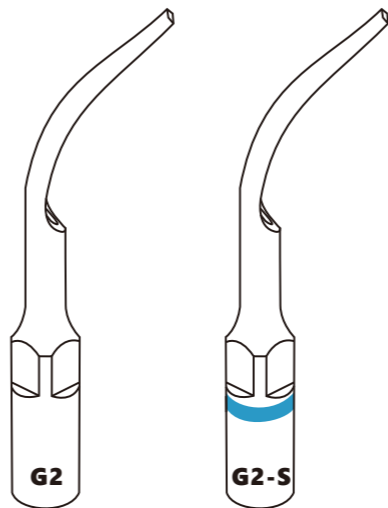

Mode: G
Power: 1-10



Recommended
Service Life
200 Teeth

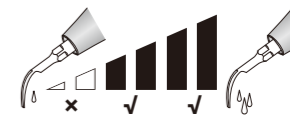

Water Flow Rate
mid&high






EMS® G4

Satelec® G4-S

Order Code 6680526

Order Code 6680619


Function Description: Used for supragingival and interdental space scaling

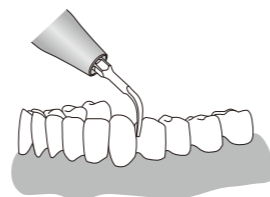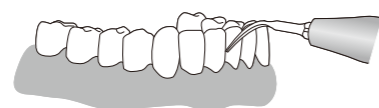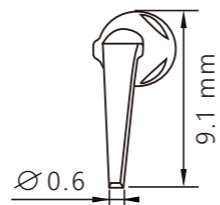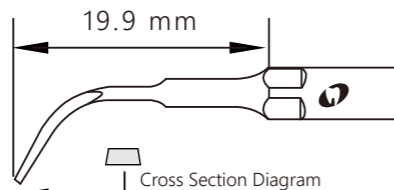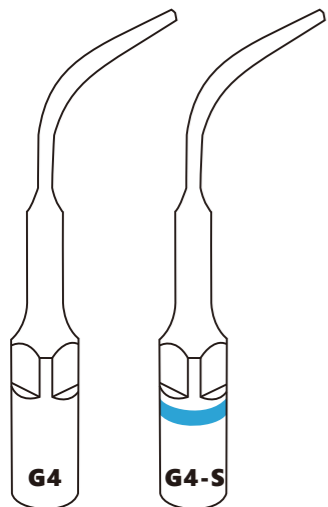

Mode: G
Power: 1-10



Recommended
Service Life
200 Teeth

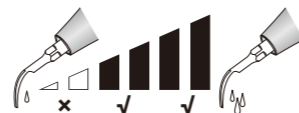

Water Flow Rate
mid&high






EMS® G5

Satelec® G5-S

Order Code 6680527

Order Code 6680620

Function Description: Removal of significant supragingival calculus

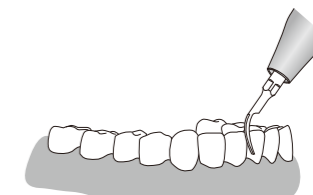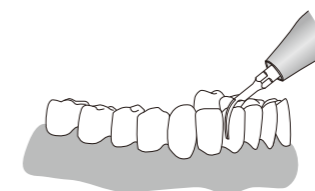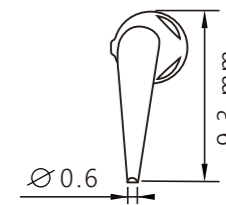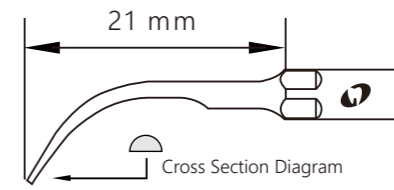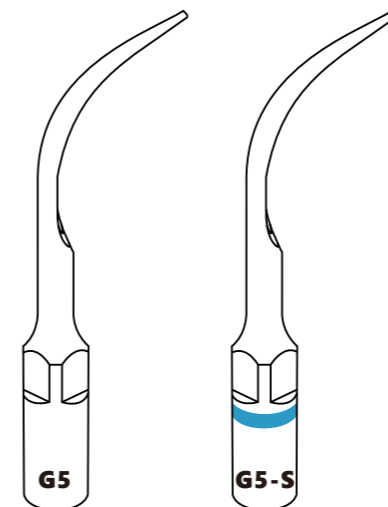

Mode: G
Power: 1-10



Recommended
Service Life
200 Teeth



Water Flow Rate
mid&high

PERIODONTIC








EMS® P1

Satelec® P1-S

Order Code 6680528

Order Code 6680621


Function Description: Removal of subgingival calculus and periodontal irrigation

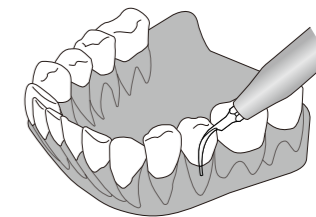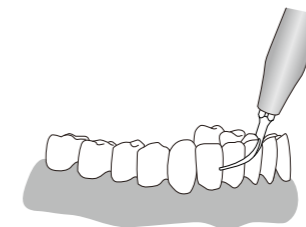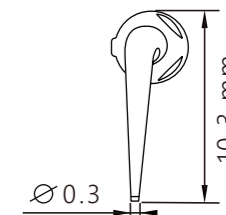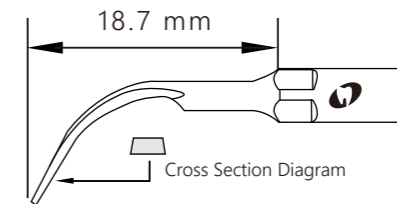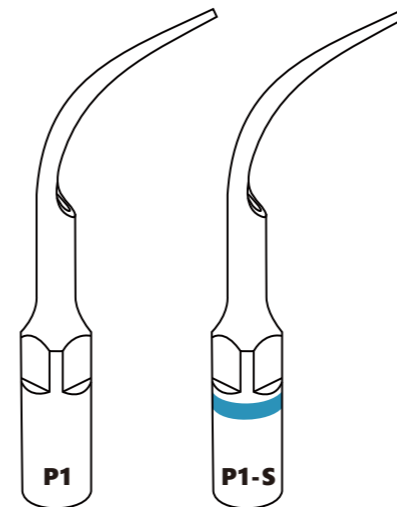

Mode: P
Power: 1-10



Recommended
Service Life
150 Teeth

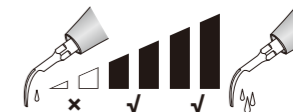

Water Flow Rate
mid&high

EMS® P3

Order Code 6680529

EMS® P4

Order Code 6680530

Satelec® P3-S











Order Code 6680622

Satelec® P4-S

Order Code 6680623

Function Description: Removal of calculus deep in periodontal pocket and its irrigation

Function Description: Removal of calculus in shallow pockets

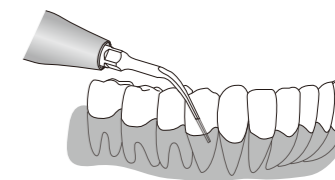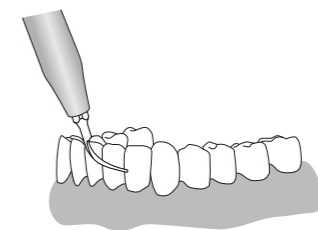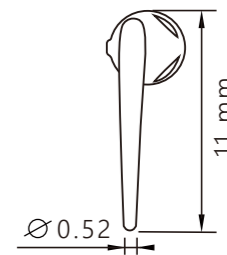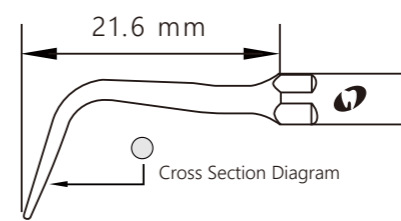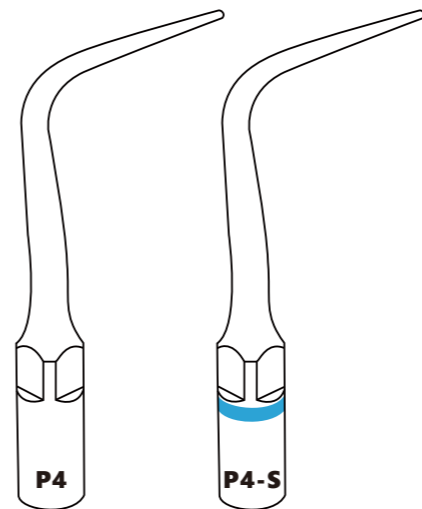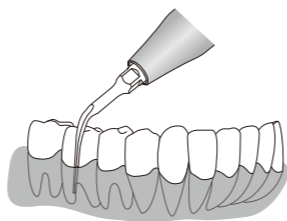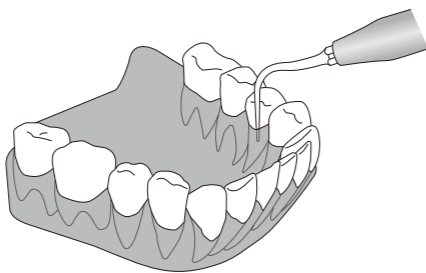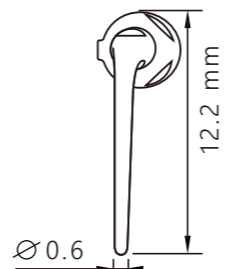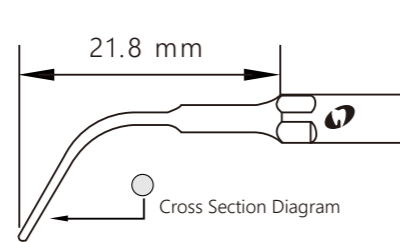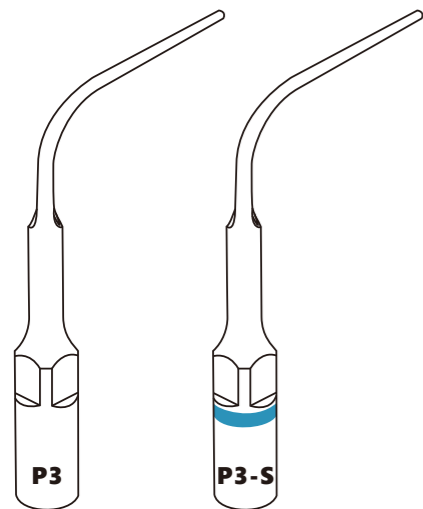


EMS® P52

Satelec® P52-S

Order Code 6680533

Order Code 6680626

Function Description: Supragingival scaling

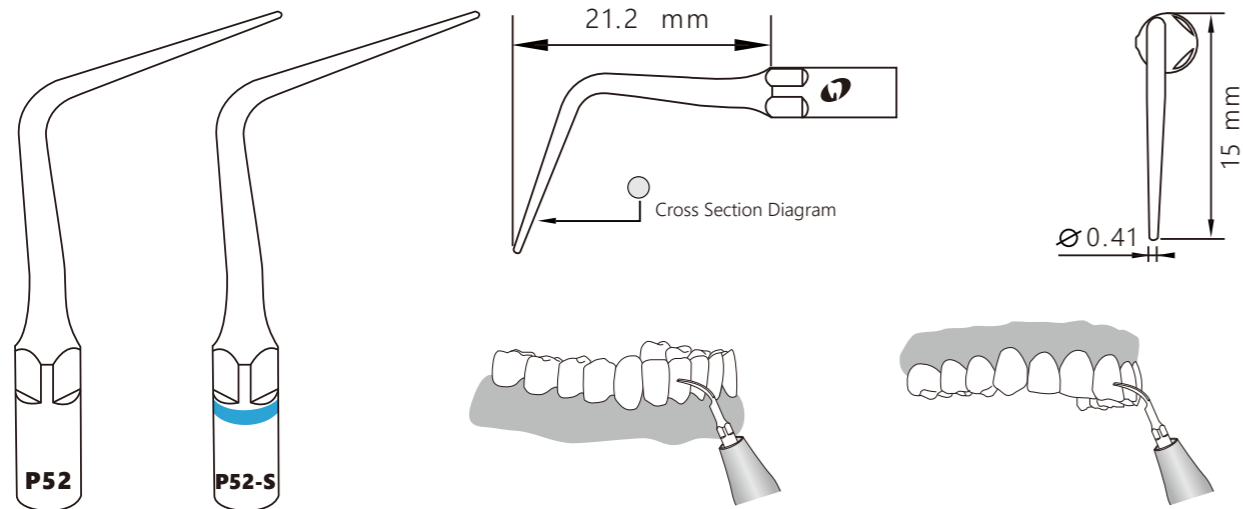



EMS® P50L

Satelec® P50L-S

Order Code 6680531

Order Code 6680624

Function Description: Used for subgingival calculus probing, subgingival proximal surface scaling and root planing

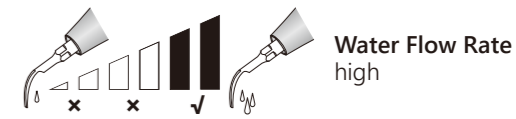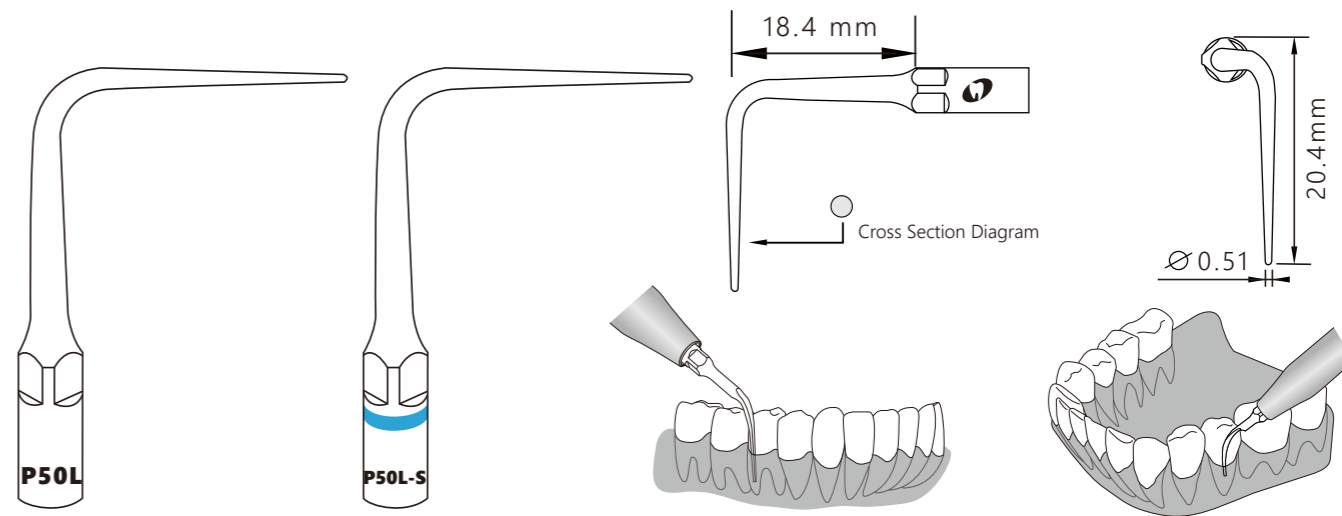


EMS® P50R

Order Code 6680532

Satelec® P50R-S

Order Code 6680625

Function Description: Used for subgingival calculus probing, subgingival proximal surface scaling and root planing

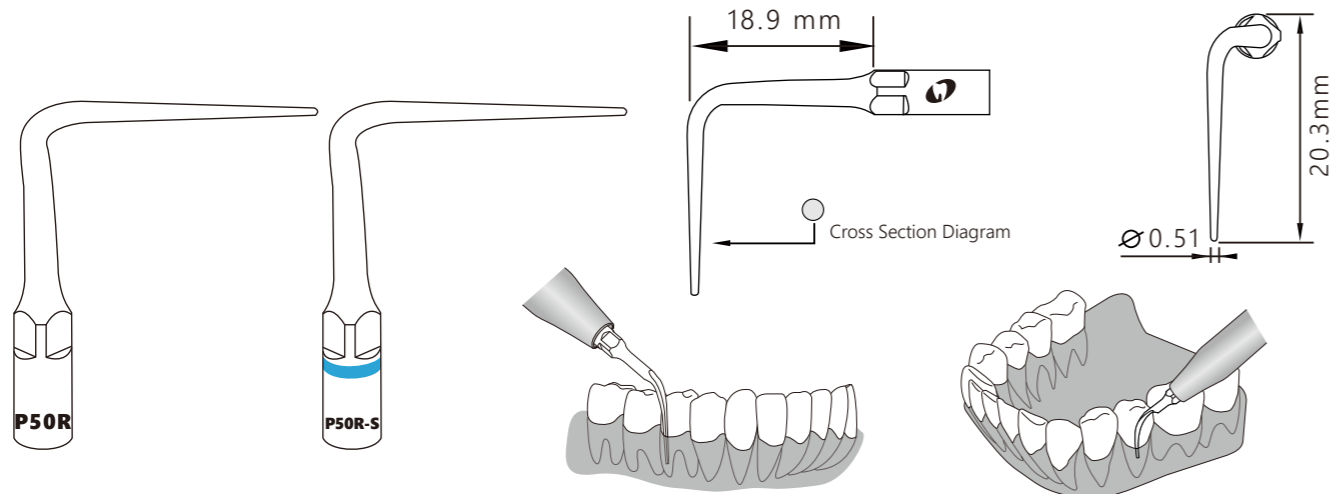


EMS® P53L

Order Code 6680534

Satelec® P53L-S

Order Code 6680627

Function Description: Used for subgingival calculus probing, subgingival proximal surface scaling and root planing

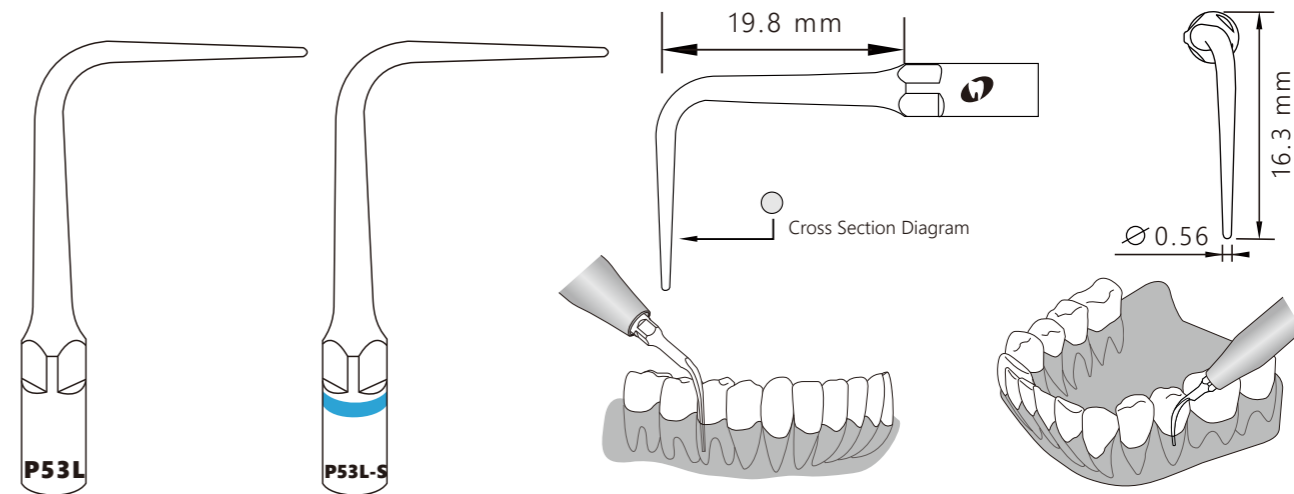


EMS® P53R

Order Code 6680535

Satelec® P53R-S

Order Code 6680628

Function Description: Used for subgingival calculus probing, subgingival proximal surface scaling and root planing

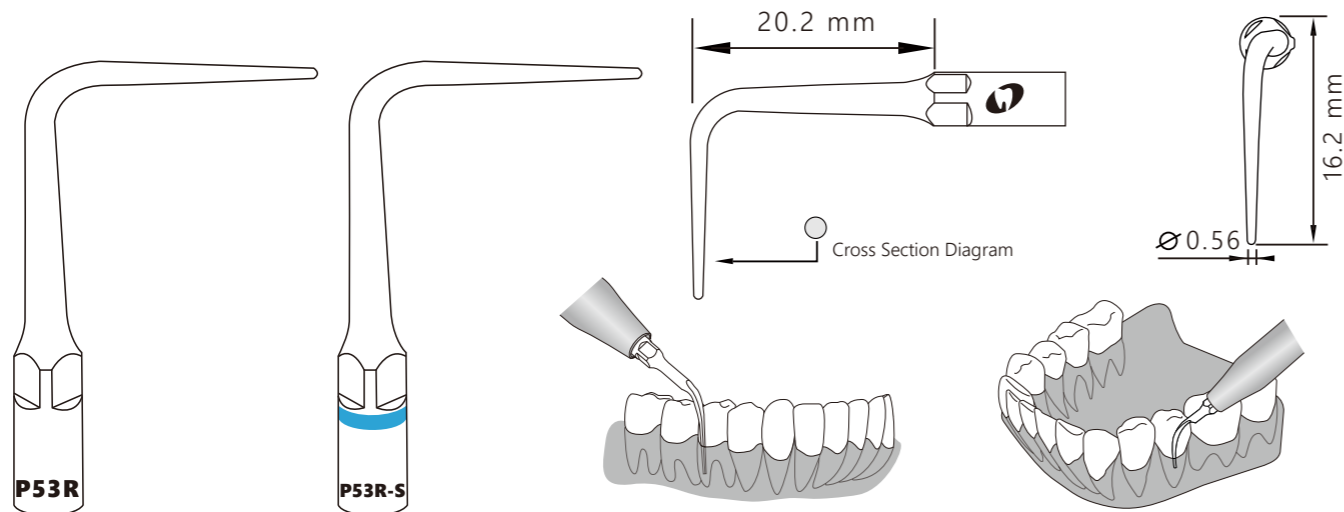


EMS® P59

Order Code 6680536

Satelec® P59-S

Order Code 6680630

Function Description: Used for subgingival calculus probing and scaling

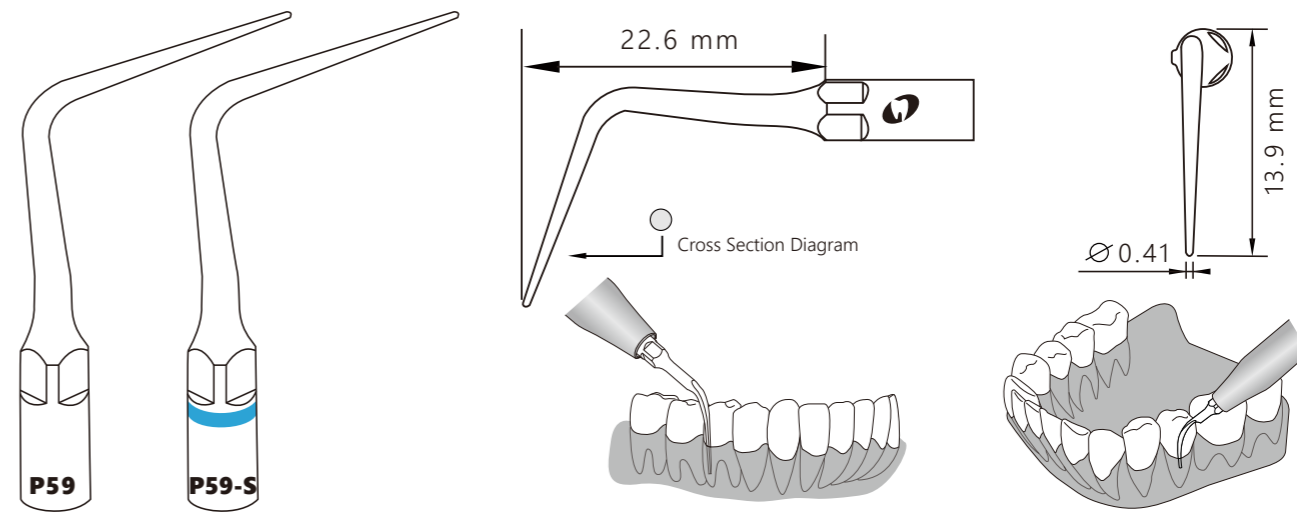






EMS® P3D

Order Code 6680654

Satelec® P3-SD

Order Code 6680636


Function Description: Used to level off the surface of endo after the periodontal flap surgery

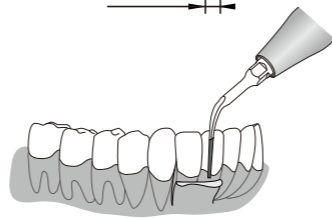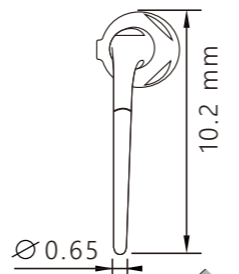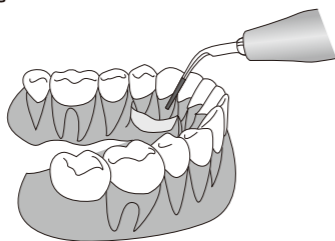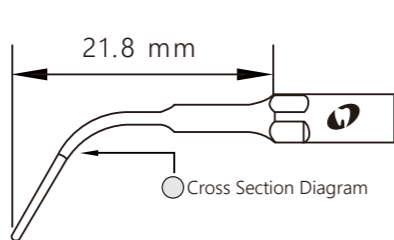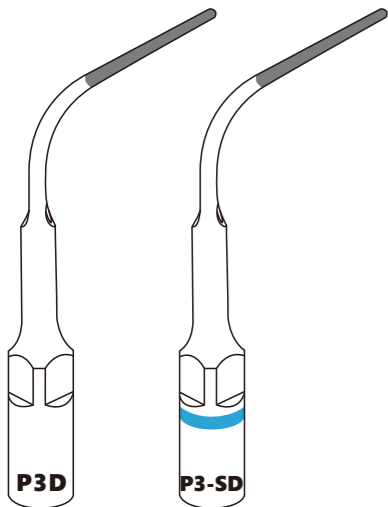

Mode: P
Power: 1-6



Recommended
Service Life
50 Teeth

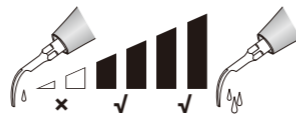

Water Flow Rate
mid&high





EMS® P4D

Order Code 6680631

Satelec® P4-SD

Order Code 6680637


Function Description: For locating root canal and removal of calcification on the one-third of crown

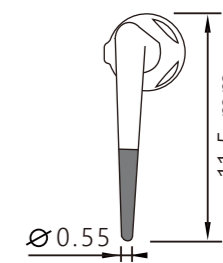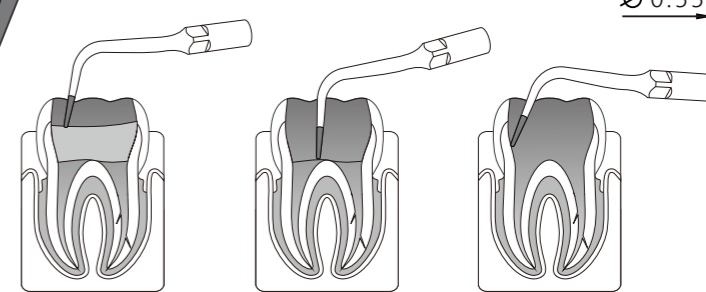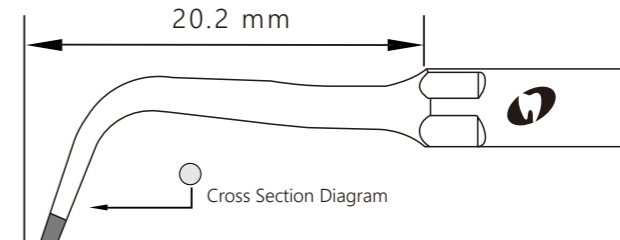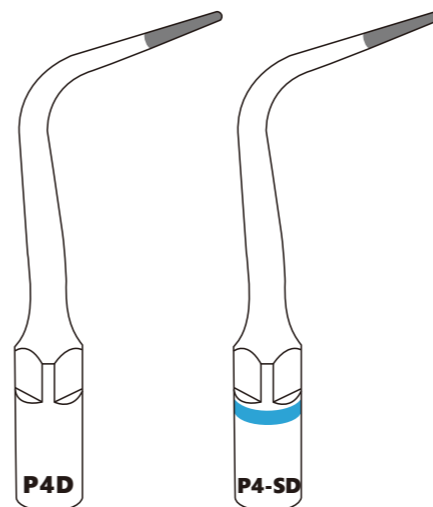

Mode: P
Power: 1-6



Recommended
Service Life
50 Teeth

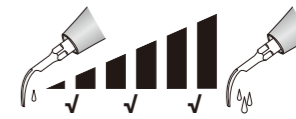

Water Flow Rate
low,mid&high



ENDODONTIC



EMS® E3

Satelec® E3-S

Order Code 6680538

Order Code 6680657

Function Description: For lateral condensation of gutta percha by heating effect

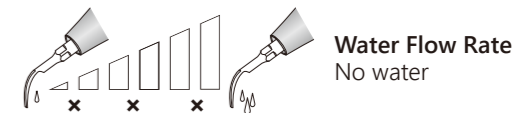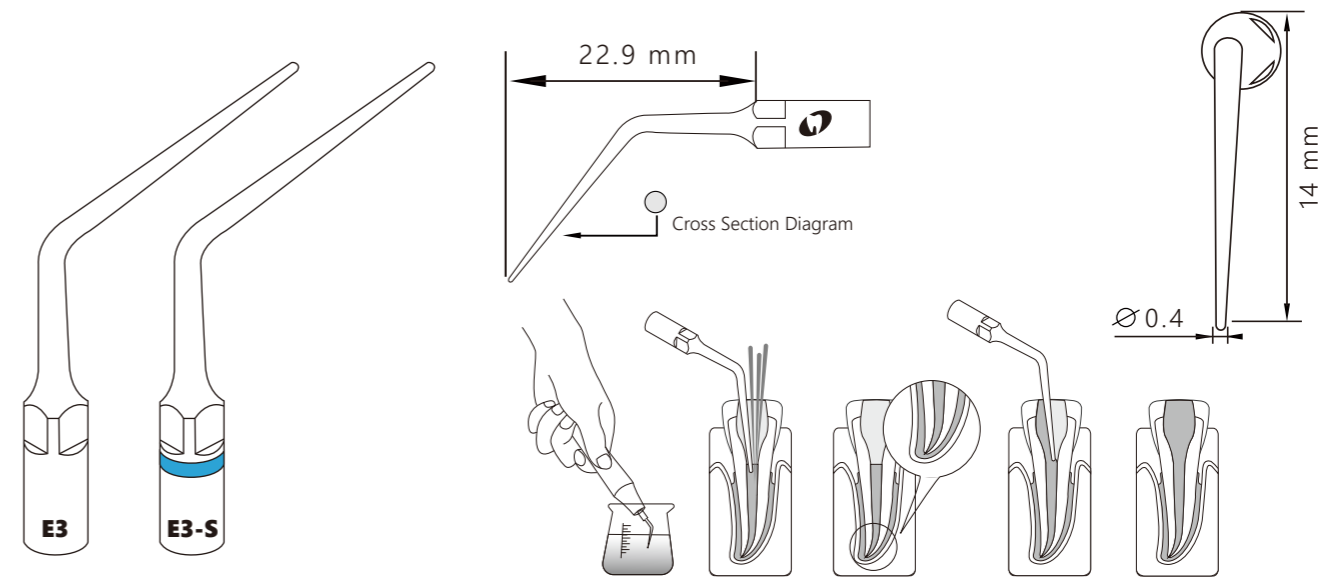


EMS® E4

Order Code 6680539

Satelec® E4-S

Order Code 6680658

Function Description: Removal of foreign filling material during re-treatment

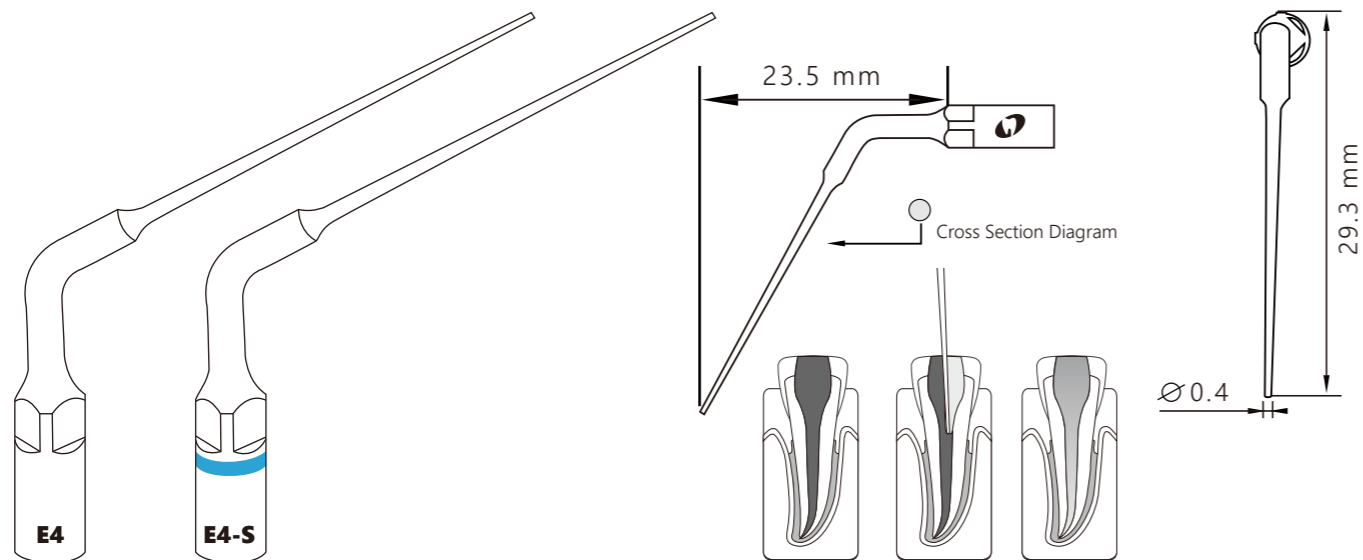

Mode: E
Power: 1-6



Recommended
Service Life
100 root canals



Water Flow Rate
No water


EMS® E5

Order Code 6680540

Satelec® E5-S

Order Code 6680659


Function Description: Removal of foreign filling material during re-treatment

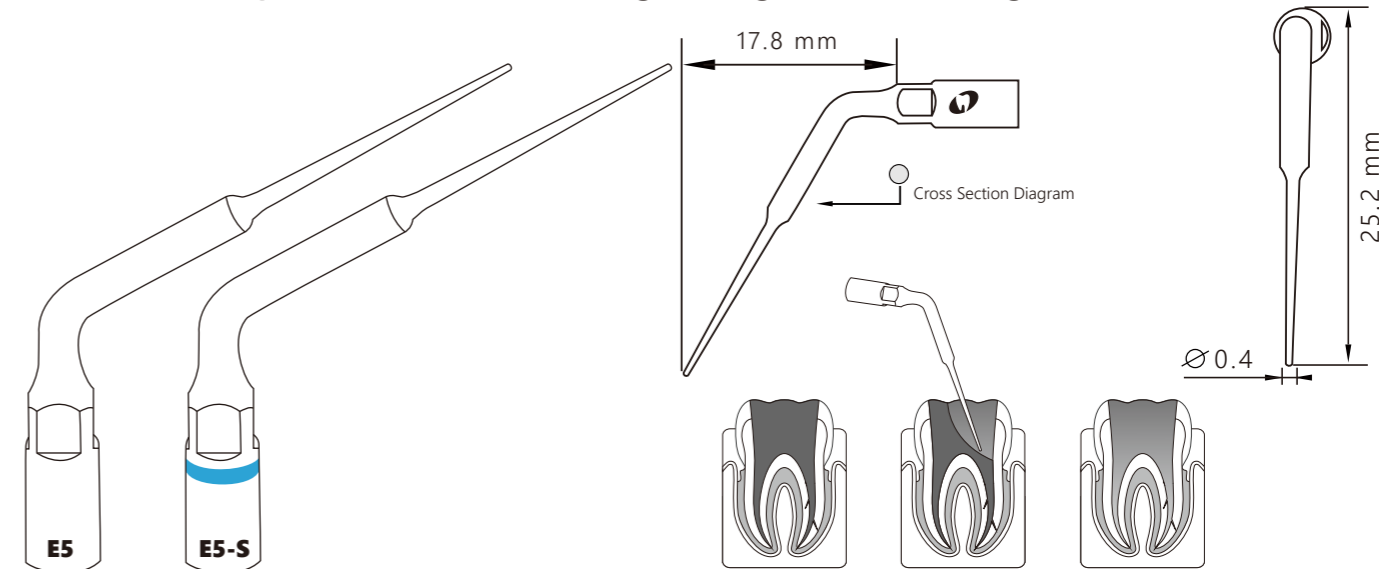

Mode: E
Power: 1-6



Recommended
Service Life
100 root canals

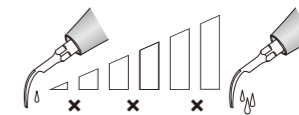

Water Flow Rate
No water


EMS® E7

Satelec® E7-S

Order Code 6680542

Order Code 6680660


Function Description: Removal of broken instrument

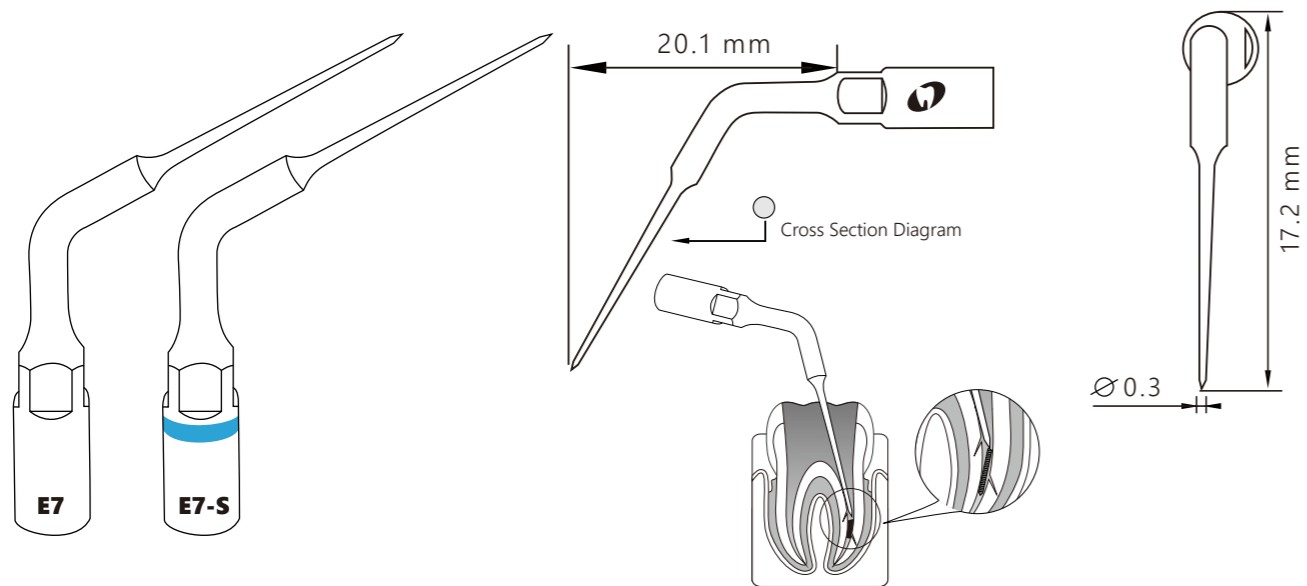

Mode: E
Power: 1-3



Recommended
Service Life
50 root canals

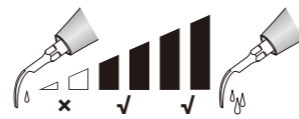

Water Flow Rate
mid&high


EMS® E9

Satelec® E9-S

Order Code 6680609

Order Code 6680615


Function Description: Flexible, used for irrigation

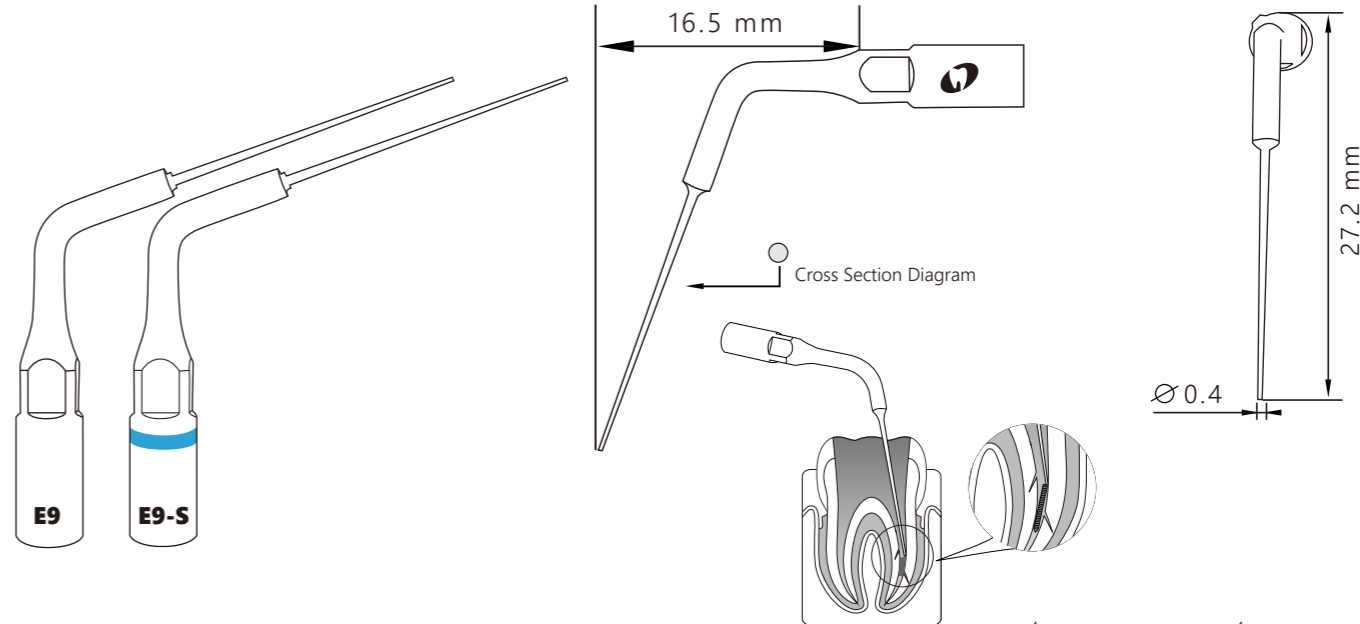

Mode: E
Power: 1-3



Recommended
Service Life
500 root canals

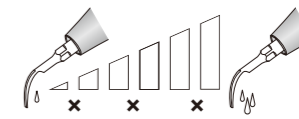

Water Flow Rate
No water



EMS® E30

Satelec® E30-S

Order Code 6680611

Order Code 6680661

Function Description: Removal of deep foreign particles in canal

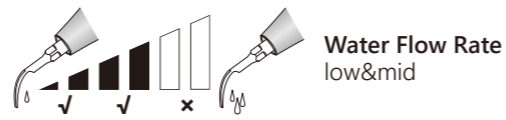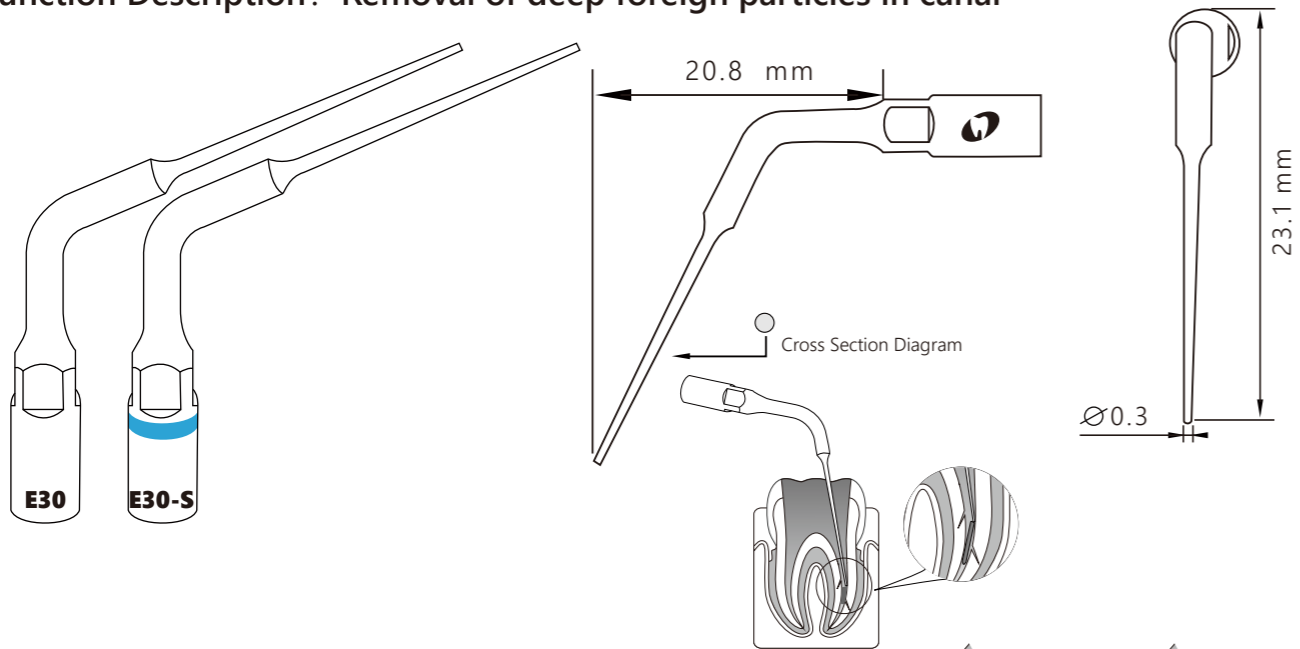



EMS® E31

Satelec® E31-S

Order Code 6680612

Order Code 6680662

Function Description: Removal of foreign particles in one-third of the canal

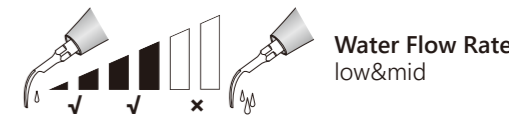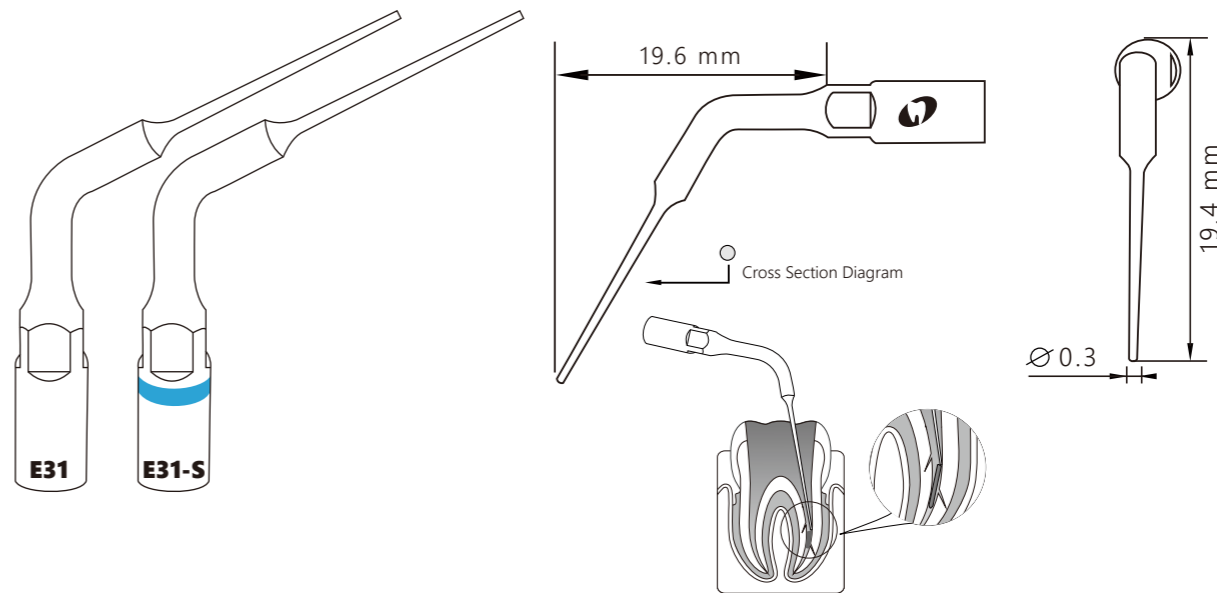


EMS® E3D

Order Code 6680651

Satelec® E3-SD

Order Code 6680638

Function Description: Enlargement of the canal walls to remove the broken instrument

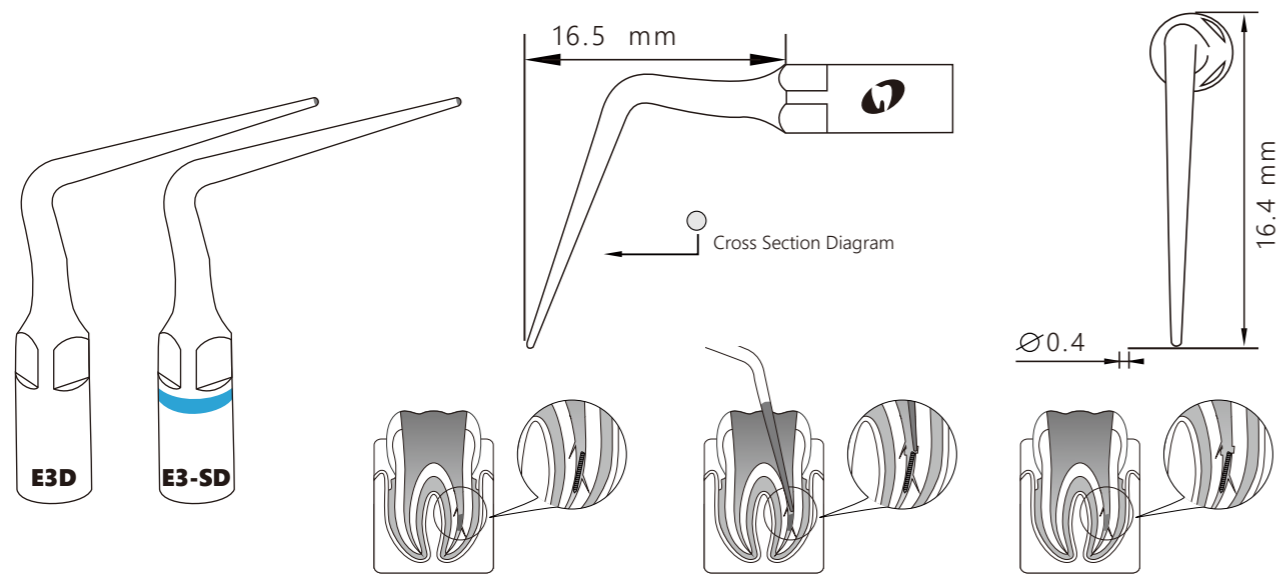



EMS® E4D

Order Code 6680652

Satelec® E4-SD

Order Code 6680639

Function Description: Removal of hard material in canal during re-treatment

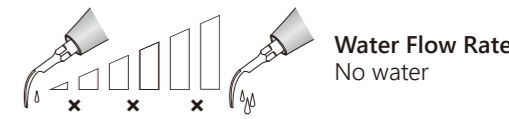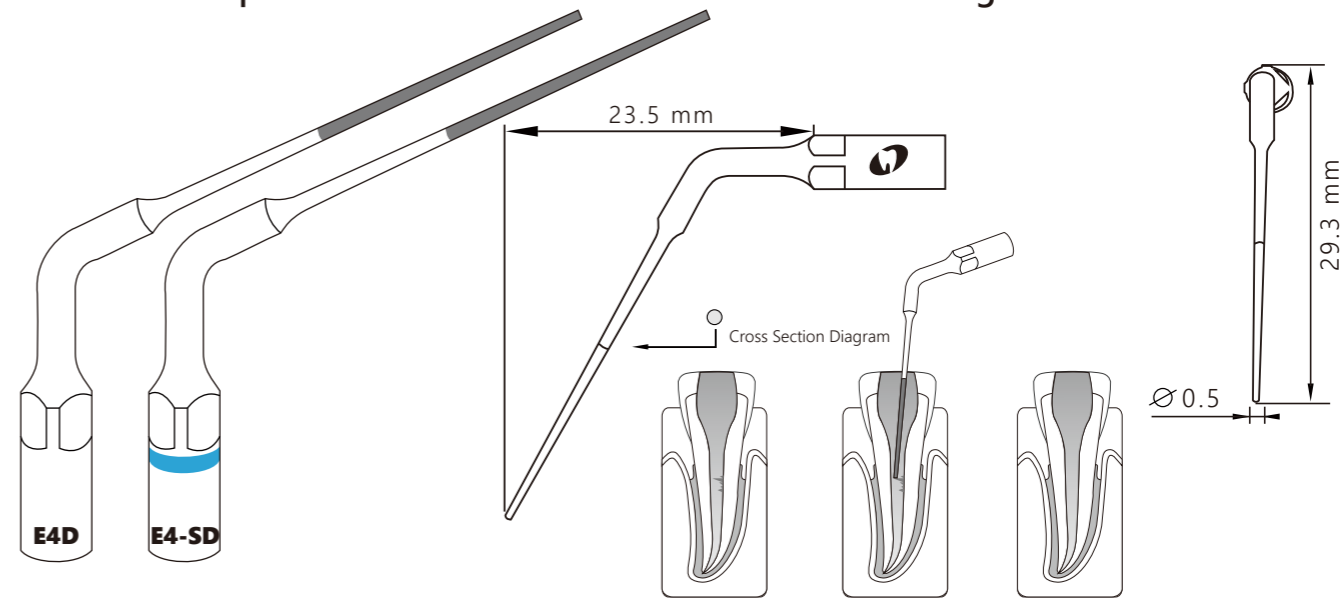


EMS® E5D

Order Code 6680653


Satelec® E5-SD

Order Code 6680640

Function Description: Removal of hard material in canal during re-treatment

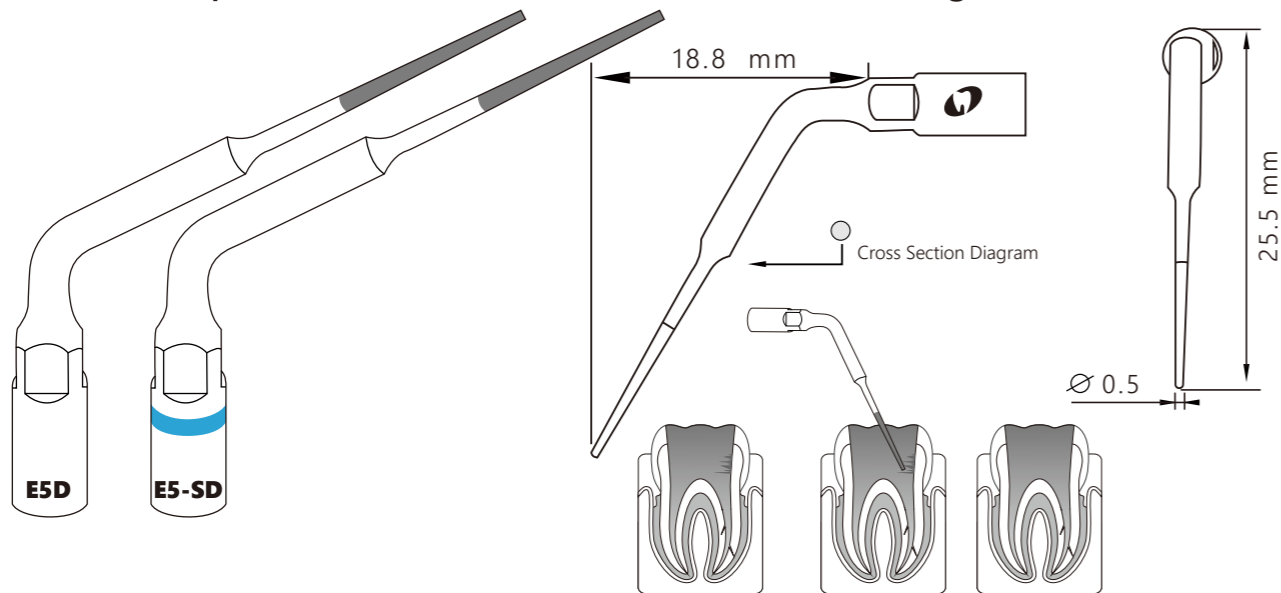

Mode: E
Power: 1-3

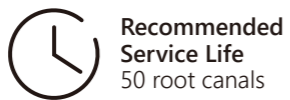

Recommended
Service Life
50 root canals



Water Flow Rate
No water


EMS® E6D

Order Code 6680632

Satelec® E6-SD

Order Code 6680641

Function Description: Removal of foreign material in cavity to access canal orifice

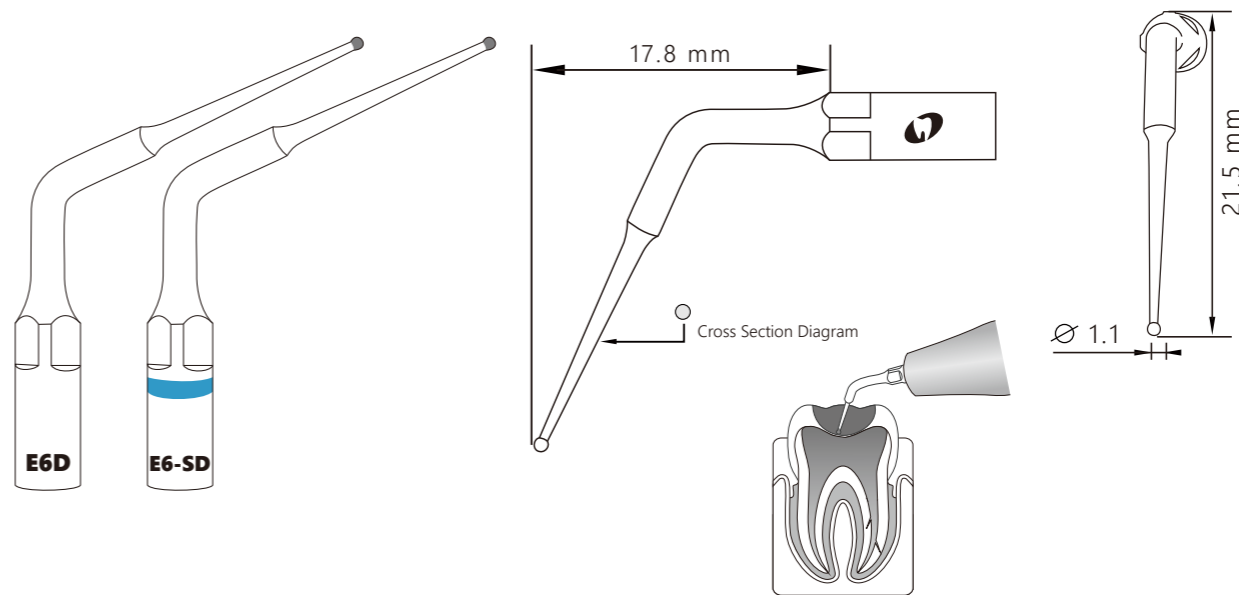

Mode: E
Power: 1-10



Recommended
Service Life
50 root canals



Water Flow Rate
mid&high


EMS® E7D

Order Code 6680633

Satelec® E7-SD

Order Code 6680642

Function Description: Removal of calcification and filling material in cavity to access canal orifice during re-treatment

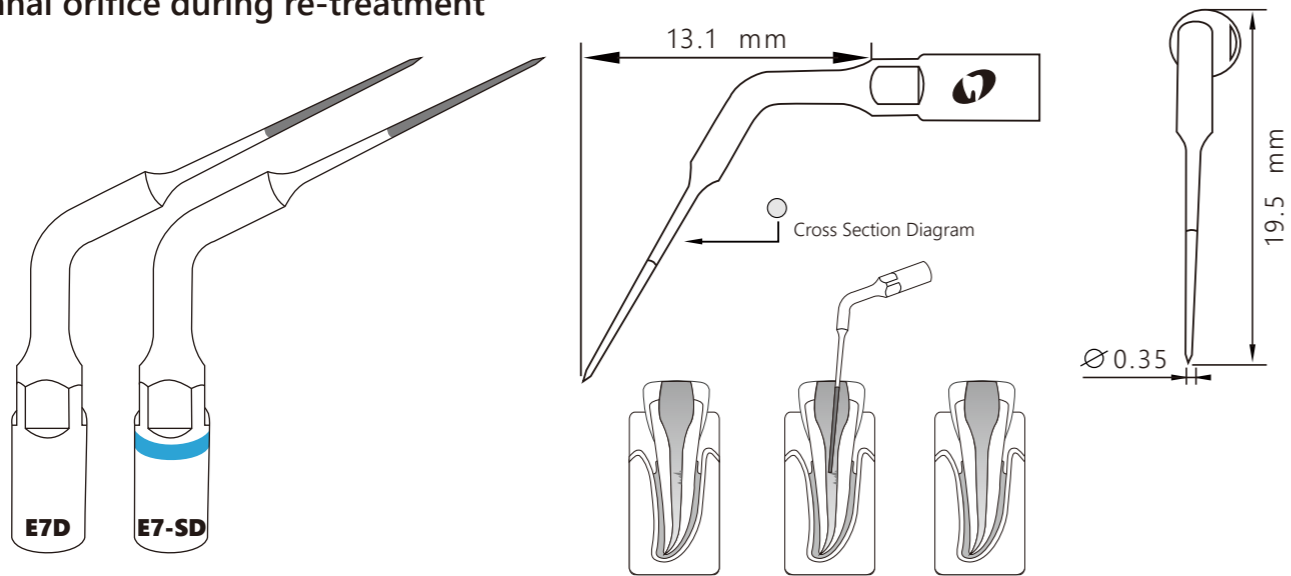

Mode: E
Power: 1-3



Recommended
Service Life
30 root canals



Water Flow Rate
mid&high

CAVITY
PREPARATION





EMS[®] TC1D

Order Code 6680648

Satelec[®] TC1-SD

Order Code 6680650


Function Description: Removal cervical caries and caries on the occlusal surface

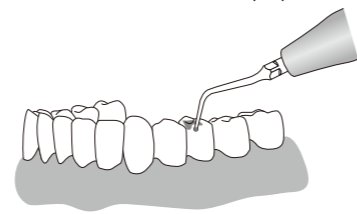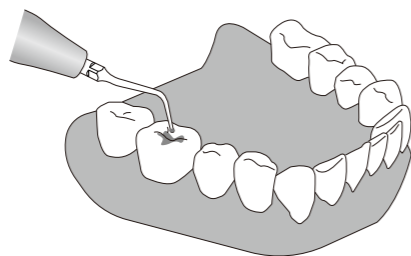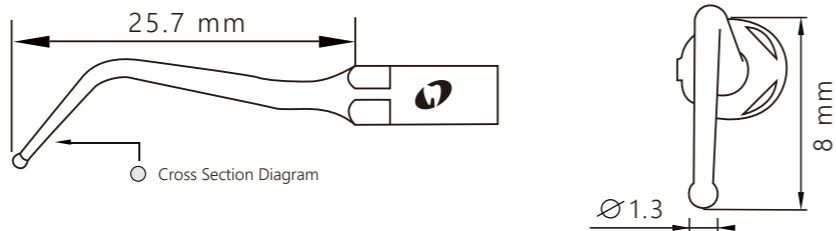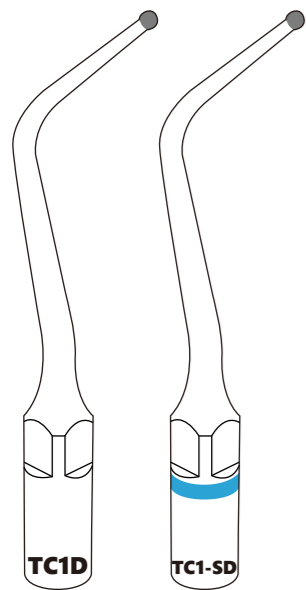

Mode: P
Power: 1-10



Recommended
Service Life
30 Teeth

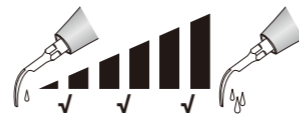

Water Flow Rate
low, mid & high